

# ALLES SPÜLT

## INSTRUCTIONS FOR THE PCR TEST AT SCHOOL

[www.bmbwf.gv.at/allesspuelt](http://www.bmbwf.gv.at/allesspuelt)

### IMPORTANT

DO NOT BRUSH YOUR TEETH,  
USE CHEWING GUM, DRINK, EAT OR SMOKE  
FOR 30 MINUTES BEFORE DOING THE TEST.




TEST TUBE  
WITH SOLUTION



QR CODES



PROTECTIVE POUCH

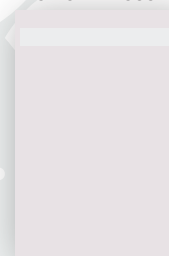

### COVID-19 DETECTED

THE SCHOOL WILL INFORM THE PUPIL,  
THE PUPIL'S PARENTS/GUARDIANS AND  
THE LABORATORY.

### COVID-19 NOT DETECTED

CONGRATULATIONS, STAY HEALTHY  
AND KEEP ATTENDING SCHOOL!

**1**

- Remove a QR code from the sheet
- Stick it on top of the test tube in the middle of the green lid

**Important**

- Press the sticker **FIRMLY** onto the clean/dry cap (not onto the tube)
- Rub firmly across the sticker – including the edges

**2**

- Open the test tube
- Empty the liquid into your mouth
- Swish the liquid around the whole of your mouth for 30 seconds

**3**

- Spit the liquid into the test tube

**4**

- Seal the test tube with the cap

**Important**

- Screw the cap on **TIGHTLY**

**5**

- Put the sample in the protective pouch and seal it well

**6**

- Place the sample in the bag