


ALLES SPÜLT

INSTRUCTIONS FOR THE PCR TEST AT SCHOOL

For more information about ALLES SPÜLT, visit: www.bmbwf.gv.at/allesspuelte

IMPORTANT

DO NOT BRUSH YOUR TEETH,
USE CHEWING GUM, DRINK,
EAT OR SMOKE FOR 30 MINUTES
BEFORE DOING THE TEST.

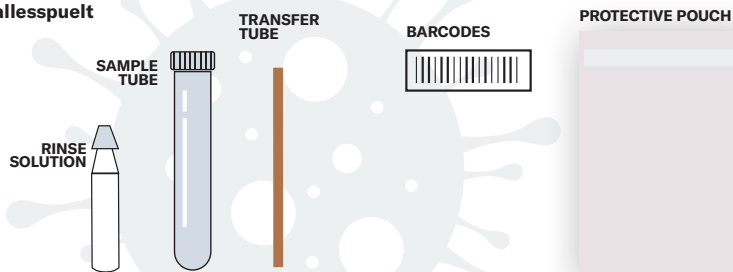

COVID-19 DETECTED
THE SCHOOL WILL INFORM THE PUPIL,
THE PUPIL'S PARENTS/GUARDIANS AND
THE LABORATORY.

COVID-19 NOT DETECTED
CONGRATULATIONS, STAY HEALTHY
AND KEEP ATTENDING SCHOOL!

1

- Remove a barcode from the sheet
- Stick it along the length of the tube (DON'T wrap it around the tube)
- Open the sample tube and place it in front of you on the table

2

- Open the container with the rinse solution
- Empty the liquid into your mouth
- Swish the liquid around the whole of your mouth for 1 minute

3

- Spit the liquid through the transfer tube into the sample tube

4

- Seal the sample tube with the cap

5

- Put the sample in the protective pouch and seal it well

6

- Place the sample in the bag